

Battelle CCDS™¹ Process

HOW IT WORKS Battelle CCDS Critical Care Decontamination System™


BATTELLE

Your Facility's Responsibility


- Your Battelle POC will provide you with 3-digit codes for each facility
- The 3-digit code must be written on each individual N95



- Collect all N95 respirators into a single plastic bag
- Once the plastic bag is filled, tie off the bag and put it into another plastic bag

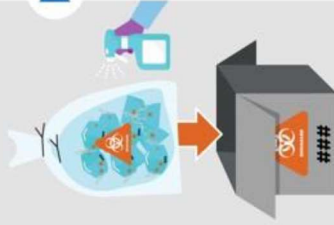

- Clean the outside bag with disinfectant
- Shipping box must be labeled with the 3-digit code and a biohazard sticker

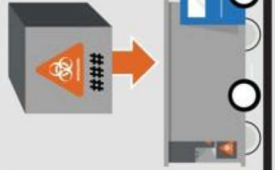

- You can either use a logistics provider of your choice or Battelle's preferred logistics provider



- Your shipments are barcoded to ensure chain of custody
- Your N95 respirators are exposed to the vapor phase hydrogen peroxide process
- After processing, your N95 respirators are verified to ensure they are free of decontaminant



- The logistics provider returns your decontaminated N95 respirators back to your facility

Battelle CCDS™ Process